SIMPLEE®

All you need. No more. But no less.

SECTIONAL MATRIX SYSTEM

(en)

Reprocessing of reusable products

Disinfection, Cleaning & Sterilisation Instructions

Basic Principles

• Simplee Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use.

• Simplee Products must be disinfected, cleaned and sterilized prior to each use.

• Effective cleaning is an indispensable requirement for proper Simplee Products sterilization.

• Consider the legal provisions valid for your country as well as to the hygienic instructions of the Dental Practice.

• The user is responsible for the sterility of Simplee Products. Therefore, please ensure that only validated procedures are used for disinfection, cleaning and sterilization. The sterilization equipment must also be maintained and checked regularly, as well as the validated parameters applied to each disinfection, cleaning and sterilization cycle.

• Materials may alter over time. Sterilization or exposure to chemicals can accelerate this deterioration. Always check your Product for wear and damage before use.

• Avoid contact of dissimilar metals at any point during the disinfection, cleaning and sterilization process.

Protection of Staff Members

Used and contaminated Simplee Products must be handled with appropriate Personal Protective Equipment.

1. Pre-treatment

Before processing Simplee Products remove coarse impurities on the products immediately after application. Do not use a metal brush. A metal brush will cause scratches or damages on the products surface.

In case of using a thermal disinfectant the steps in this guidance (see below) "2. Disinfection" and "3. Rinsing" are unnecessary, provided that the dirty Simplee Products have not been stored dry longer than 2 hours.

2. Disinfection

• Do not leave dirty Simplee Products dry – put them into a disinfecting liquid as soon as possible.

• We suggest to use an enzymatic pre-cleaner disinfectant. Or a pre-cleaning disinfectants based on Quaternary Ammonium compounds.

• Follow carefully the instructions for use given by the manufacturer of the selected disinfectant. Please note that too strong concentration of the disinfectant or too long exposure time may damage the products.

• Change the disinfectant regularly according to the instructions.

3. Rinsing

• Immediately after the disinfection, Simplee Products must be rinsed carefully under running lukewarm water.

4. Cleaning

Always clean Simplee Products, disinfection and rinsing are not sufficient.

Cleaning alternatives:

Manual (A), Ultrasonic bath (B) Thermal Disinfectant (C)

If possible, an automatic procedure in a thermal disinfectant should be used for cleaning of Simplee Products.

A manual procedure, even in case of application of an ultrasonic bath, should only be used if an automatic procedure is not available or if such a method is not compatible with specific materials; in this case, the significantly lower efficiency of a manual procedure must be considered.

A. Manual Cleaning

• Completely disassemble Simplee Products if applicable

• We suggest to use a cleaning or cleaning solutions based on Quaternary Ammonium compounds. Check the pH-value of the cleaning solutions; it must be as neutral as possible.

• Follow carefully the instructions for use given by the manufacturer of the selected cleaning solutions. Please note that too strong concentration or too long exposure time may damage Simplee Products.

- Change the cleaning solutions regularly according to the instructions.
- Always rinse carefully Simplee Products under running water after cleaning.

B. Ultrasonic bath

• Please assure that Simplee Products have the relevant symbol on the packaging.

• Please assure that Simplee Products have the relevant symbol on the packaging.

• Completely disassemble Simplee Products if applicable.

• Follow carefully the instructions for use of the ultrasonic bath device.

• Service and clean the device frequently, according to the instructions for use.

• Ensure that maximum load is not exceeded.

• We recommended to use a cleaning solutions such as - or cleaning solutions based on Quaternary Ammonium compounds. Check the pH-value of the cleaning solutions; it must be as neutral as possible.

• Follow carefully the instructions for use given by the manufacturer of the selected cleaning solutions. Please note that too strong concentration or too long exposure time may damage Simplee Products.

• Change the cleaning solutions regularly according to the instructions.

• Remove the devices from the ultrasonic bath immediately after cleaning.

• Always rinse carefully Simplee Products under running water after cleaning.

C. Thermal Disinfectant

• Please assure that Simplee Products have the relevant symbol on the packaging.

• Completely disassemble Simplee Products if applicable.

• Follow carefully the instructions for use given by the manufacturer of the thermal disinfectant and of the selected program; and prefer cleaning agents with corrosion protector.

• Inspect the device regularly according to the instructions for use. Service and clean the device frequently, according to the instructions for use.

• Ensure that maximum load is not exceeded.

• Ensure that the selected washing programme and cleaning agents are effective enough.

• Remove Simplee Products from the thermal disinfectant immediately after cleaning.

5. Drying

Before sterilizing in the autoclave, Simplee Products must be thoroughly dried.

The normal tap water may contain different chemical ingredients which may cause chemical reactions when autoclaving.

6. Checking

After cleaning, check that Simplee Products are clean by rubbing the surface with an eraser or finger.

WARNING! It is extremely important to check that the products are truly clean and dry before the sterilizing procedure.

7. Sterilization

Sterilize Simplee Products **only** in a steam autoclave using distilled water and at the temperature specified on the relevant symbol. **Do not use chemical or cold sterilization.**

• Please assure that Simplee Products have the relevant symbol on the packaging.

• Follow carefully the instructions for use of the autoclave.

• Inspect the device regularly according to the instructions for use. Service and clean the device frequently, according to the instructions for use.

• Ensure that maximum load is not exceeded.

• Place all the medical device components in an autoclave tray, or in a sterilization disposable pouch (recommended).

• Plastic parts must not touch the autoclave walls, since the temperature there might be higher.

• We recommend to use always the following cycle (Prion Cycle):

Temperature of sterilisation: 134°C

Pressure of sterilisation: 2 bar

Time of sterilisation: 20,5 min

Drying time: 20 min

• Remove Simplee Products from the autoclave immediately after sterilisation.

8. Storage

• Store sterilized product in a dry and dust free location.

• Keep product sealed in the sterilization pouch until ready to use.

DISCLAIMER:

This high quality product is manufactured by Polydentia SA, in Mezzovico, Switzerland.

The content of this packaging has been inspected, tested and approved by our quality officers.

Materials used during our manufacturing process are designed to provide durability and performance.

Polydentia constantly looks for the highest quality plastics for its product; nevertheless please take into account that plastics may alter over time. Sterilization or exposure to chemicals can accelerate this deterioration. For the longest lifetime of your premium quality product always clean and sterilize it following these instructions.

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