SIMPLEE®

All you need. No more. But no less.

SECTIONAL MATRIX SYSTEM

en

Instructions for use

Description

Simplee Sectional Matrix System: Complete sectional matrix system developed to offer gentle tooth separation in order to create contact points in the restoration of class II cavities. This kit features a classic sectional matrix ring made of nickel titanium. It is reinforced to achieve an optimal and gentle separating force in combination with interdental wedges made of maple wood. For the restoration of very extensive cavities, the kit also includes the newly re-designed silicone tips. These anatomically shaped silicone tips ensure optimal matrix retention and adaptation to the tooth's anatomy. Sectional matrices complete the kit's assortment.

Composition

Simplee Sectional Matrix System: NiTi Ring with thermoplastic overmolding
Delta Tubes: silicone
Sectional matrices: Stainless steel

Indications

Simplee Sectional Matrix System is indicated for gentle tooth separation in order to create correct contact points in the restoration of class II cavities.

Contraindications

None Known.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to the Simplee Sectional Matrix System Kit and its component is necessary.

Shelf life

Please refer to the expiration date on the packaging

Preservation and storage

Store in a clean, dry environment away from moisture or excessive humidity.

Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. Always ensure the rings using dental floss. In case of accidental inspiration of a Simplee Sectional Matrix System component (matrix, retaining ring and its extremities, wedge) seek immediate medical attention.

In case of ingestion of a Simplee Sectional Matrix System component, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately.

All reusable Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use. Please refer to the Cleaning, disinfection and Sterilization Instructions at www.simpleedental.com.

Application

See the following pictograms.

Disposal Because of their composition, Matrices can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.

Ring reconditioning

For a proper, effective and safe reconditioning of the Simplee Sectional Matrix System and their extremities read the Cleaning, Disinfection and Sterilization Instructions at www.simpleedental.com.

Application

1. Prepare the cavity

Remove the carious lesion and prepare tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed.


2. Place the matrix

Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers.


3. Wedging

Select the appropriate sized wooden wedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging.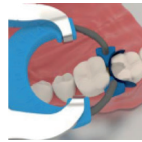

4. Place Simplee NiTi Ring

Expand Simplee NiTi Ring Classico using a Forceps or standard rubber dam forceps. Place the ring carefully and securely. Please refer to the Tips & Tricks section to see the different ways to position Simplee NiTi Ring and ensure best teeth separation as well as matrix adaptation.

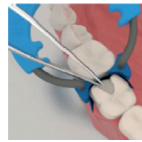5. Burnish the matrix

For an excellent contact burnish the band against the adjacent tooth using a ball burnisher or tweezers

6. Restore the cavity

Restore the cavity according to the filling material manufacturer's instructions.

7. Remove the sectional matrix system

Remove Simplee NiTi Ring, the wedge and the matrix. Our suggestion for an easy matrix removal is to use a Forceps.

8. Finishing and polishing

With thin burs remove any small excess of material then polish the restoration according to the finishing and polishing instruments manufacturer's recommendations.

Tips & tricks using Simplee Sectional Matrix System

Wedging

According to the clinical situation, the wooden wedge can be broken in order to better accommodate the NiTi Ring with silicone tips. This trick is very helpful in case of small teeth.

Silicone tips placement options


Silicone tips can be oriented according to the clinical situation to obtain the best matrix retention and adaptation to the tooth's anatomy. The most common cases are explained as follows (Case A, B). Different orientation possibilities can be combined to best adapt the NiTi Ring to each specific clinical situation. The use of silicone tips is not mandatory, NiTi Ring can be placed also without Silicone tips (Case C, D).

Case A, small cavities

silicone tip edges allow a perfect adaptation between the teeth

Case B, broad cavities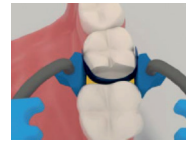

silicone tips flat surfaces provide support on the adjacent tooth, no matrix deformation

Case C and D, NiTi ring without silicone tips


NiTi Ring lines placed near to the wood wedge This placement technique is suitable for cases of a wide space in the interproximal area.


Simplee Sectional Matrix System lines placed over the wood wedge This placement technique is suitable if the interproximal space is tight or when the cavity is deep.

MOD and multiple tooth restorations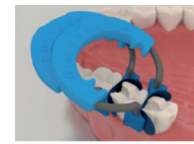

Simplee Sectional Matrix System rings are designed to be stacked over one another to perform a MOD and multiple tooth restorations.

1. Prepare the cavities as described in the instructions for use, step 1.
2. Place matrices and wedges on both mesial and distal surfaces of the tooth being restored
3. Position NiTi Ring
4. Complete the restoration as described in the instructions for use, steps 5 to 8.

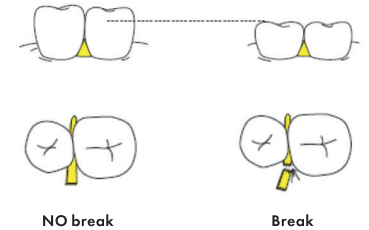NO break

Break

Symbols

	Item number
	Batch code
	Use by
	Consult instructions for use
	Do not reuse
	Caution
	Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner
	Manufacturer
	CE mark
	Ultrasonic bath
	Washer-disinfector for thermal disinfection
	Sterilizable in a steam sterilizer (autoclave) at the temperature specified
	Medical Device
	European Authorised Representative
	Importer
	Distributor

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