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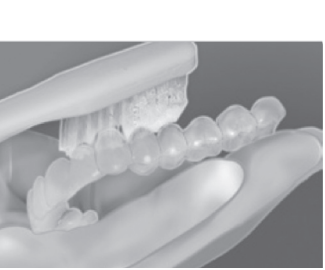
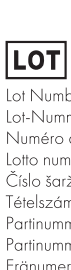
Clean the tray with a toothbrush and water after use and store in a clean dry container.

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Tooth Whitening Gel

### Description

Simplee Brighter whitening is a dentist-controlled, dentist supervised, take-home whitening system, dispensed in syringes. The products are clear, high viscosity, sticky gels, containing carbamide peroxide along with potassium nitrate. The gels are available in carbamide peroxide concentrations of 10% and 16% (3.6% and 5.8% hydrogen peroxide equivalent).

#### Precautions for Europe

- Not to be used on a person under 18 years of age.
- To be only sold to dental practitioners. For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use.
- Please note the expiration date printed on the outer package/syringe.

#### Precautions

- If you have any questions regarding the appropriate use of this product, including how long it will take to whiten your teeth, please consult your dentist.
- Regular dental checkups and cleanings are important before and after whitening to maintain a healthy smile.
- Some patients experience sensitivity with whitening. Should this occur, stop treatment, and contact your dentist.
- Some patients have noticed temporary discomfort of the gums, lips, throat, or tongue. Should any of these symptoms persist more than two days or progressively worsen, call your dentist. These side effects will usually subside within 1-3 days after treatment is discontinued.
- Pregnant or breastfeeding women, or patients with serious health concerns should not whiten. Consult their primary care provider prior to treatment.
- If patient has a known allergy or chemical sensitivity to any of the ingredients in Simplee Brighter whitening, do not use. Mint flavor can be irritating to some patients.
- Do not use household bleach to whiten teeth.
- Do not use tobacco or eat while whitening trays are in place.
- Teeth are naturally darker along the gum line. These areas may require more time to lighten than the rest of the tooth surface and usually remain slightly darker.
- White spots may appear more prominent on the enamel during the whitening process and may blend with continued whitening.
- Crowns, bridges, partial dentures, veneers, and composite fillings will not whiten.
- Coffee, tobacco, and other products can re-stain your teeth over time. Should this occur, teeth can be re-whitened.
- Some amalgam fillings may leave a dark purple color in your whitening tray; this is normal.
- Check expiration date and store according to instructions.
- Avoid swallowing large amounts of gel or water used to rinse gel residue from the teeth.
- Avoid contact with eyes. Rinse immediately if product comes into contact with them.

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