SIMPLEE® Mixing Gun 1:1/2:1

Stand: 150525

Instructions for use

Medical Device- only for use by dental health care professionals

1.1 Intended Purpose

Mixing gun for automatic mixing and direct application of silicone impression materials in a mixing ratios of 1:1, 2:1.

1.2 Indications

The Dispensing gun is suitable for all common dental materials (A and C silicones) in cartridges.

1.3 Contraindications


In case of known or suspected allergy to components of this product, its use is contraindicated.

1.4 Potential Side effects

Hypersensitivities to the product or its components cannot be excluded in individuals predisposed. In case of suspicion, please contact the manufacturer for ingredients information.

2.0 Dispensing Gun Operation Instructions

Install the cartridge

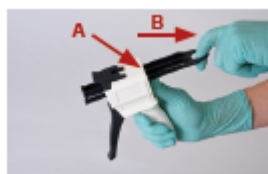PICTURE 1 Push up the lever at the back of the dispensing gun (A) and pull back the slide completely (B).

PICTURE 2 and 3 Lift the plastic lever to insert the cartridge. Important: Make sure that the notches on the cartridge match those on the mixing gun. Then lock it by pushing the plastic lever back down.

Install the Mixing Tip

PICTURE 4 To insert the Mixing Tip, remove the cartridge cap by turning the cap counterclockwise 90 degrees, and then pull it off. You can keep the cap to close the cartridge if the contents have not been used up completely.

PICTURE 5 Make sure the notches on the Mixing Tip match those on the cartridge. Press firmly to attach the Mixing Tip. Then turn the Mixing Tip 90 degrees in a clockwise direction to its "locked" position. The mixing gun is now ready for use.

After dispensing

PICTURE 6 and 7 To remove the cartridge, press the mixing gun release lever upwards. (A) Completely retract the lever. (B) Place the cartridge lock in the up position and lift the cartridge out of the dispenser.

3.0 General Notes

Please carefully observe the following safety instructions and all processing in other sections. This product is intended to be used only in accordance with this instructions for use. Any use of this product inconsistent with these instructions is at the discretion and sole responsibility of the processing dental health care professional. Commercially available cartridges fit exactly into Dispensing gun. Make sure that they are securely locked in the Dispensing gun.

3.1 Warnings and Safety Information

Apply the cartridge content by evenly activating the lever. Excessively fast extrusion of the contents from the cartridge or the exertion of extremely high force during application should be avoided. Do not use visibly damaged devices. Before each use check functionality (peace of movement). See section: Maintenance, Inspection and Testing of processing instructions. Following this procedure there are no restrictions for reuse.

3.2 Storage Conditions

When storing protect against dust and moisture.

3.3 Disposal Information

Recommendation: Dispose of in accordance with local regulations. Do not dispose of contents or partially emptied packaging in household waste or allow them to enter the sewage system.

4.0 Cleaning, Disinfection and Sterilisation

WARNING: Please pay attention to the country-specific regulation regarding the hygiene standards and cleaning/disinfection of medical devices in dental practices. Wear suitable gloves. The Dispensing gun is delivered unsterile. It must be sterilized before first use and cleaned, disinfected and sterilized after each use.

RESTRICTION OF PROCESSING: Oxidizing disinfectant solvents are not suitable. Strong acids and strong bases may oxidize the plastic material.

Instructions:

Point of use: No specific requirements

Containment and Transport:

Transport of the contaminated Dispensing gun in a closed container

Preparation for decontamination:

Personal protective equipment (gloves, water repellent protective apron, face protection mask or protective glasses)

Cleaning Manually

Equipment: mild alkaline cleaner detergent, Tap water / flowing water $20^{\circ} \pm 2^{\circ}\text{C}$ minimum drinking water quality, cleaning brush, tank or basin for detergent

- 1) The cleaner is prepared in accordance to the manufactures instructions for the detergent (max. 40°C)
- 2) Carry out all further steps below the liquid level to avoid splashing of contaminated liquid.
- 3) Immerse the Dispensing gun completely in the cleaning solvent.
- 4) Brush the hard to reach areas of the submerged Dispensing gun with a soft brush for 1 minute. Pay attention to the critical, hard to reach areas where it is not possible to assesses the cleaning efficacy.
- 5) Move all moveable components 5 times in the cleaning solvent.
- 6) Total exposure in the cleaning solvent minimum 5 minutes (5 minutes were validated).
- 7) Remove the Dispensing gun from the cleaning solvent and rinse under running tap water (minimum drinking water quality) from 1 minute to remove the cleaning solvent completely. Move the moveable components 5 times.
- 8) Inspect for cleanliness, if debris is visible, repeat the above steps.

Disinfection Manually

Equipment: instrument disinfectant, demineralized water (according KRNK/BfArM recommendation free of facultative pathogenic micro-organism) ($20^{\circ} \pm 2^{\circ}\text{C}$), disinfectant tank, lint free gauze, medical compressed air (according to the European Pharmacopoeia).

- 1) The disinfectant is prepared in accordance to the manufactures instructions
- 2) Immerse the Dispensing gun completely in the disinfection solvent. Make sure that all surfaces are completely moistened with disinfection solvent.
Move the movable components 5 times. Exposure time of the disinfectant solvent in accordance to the manufactures instructions (15 minutes were validated).
- 3) Remove the Dispensing gun from the disinfection solvent.
- 4) Rinse the Dispensing gun completely under demineralised water for minimum 1 minute to remove the disinfection solvent completely.
Move the movable components 5 times.
- 5) Wipe with a lint free gauze and / or dry with medical compressed air (according to the European Pharmacopoeia).

Cleaning and disinfection: Automated WD

Equipment: Washer disinfecter in accordance with EN ISO 15883-1-2 with thermal program (temperature $90\text{--}95^{\circ}\text{C}$), mild alkaline cleaner detergent, medical compressed air (according to the European Pharmacopoeia).

- 1) Open the cover of the retainer and place the Dispensing gun in a suitable tray or place them on the local carrier so that all inner and outer surfaces will be cleaned and disinfected.
- 2) Close the WD and start the program.

Program Step	Water	Dosage	Time	Temperature
Pro-Rinse	Cold		5 min	
Dosage cleaner		According to manufactures instructions		According to manufactures instructions
Clearing	Deionised water		10 min	55°C
Rinse	Deionised water		2 min	
Disinfection	Deionised water		According to manufactures instructions	According to manufactures instructions
Drying			15 min	Up to 120°C

¹ Public authorities may adopt other implemented provisions (parameter for disinfection performance) within their field of Responsibility

- 3) At the end of program, remove the Dispensing gun.
- 4) Check for dryness. If it is necessary, wipe with a lint free gauze and / or dry with medical compressed air (according to the European Pharmacopoeia).
- 5) After removal from the WD, inspect the Dispensing gun for cleanliness. If debris is still visible, clean the Dispensing gun manually.
Afterwards the automatic cleaning process must be carried out again.

Maintenance, Inspection and Testing

All Dispensing guns must be visually checked for cleanliness, dryness and functionality, if necessary using an illuminated magnifier (3-6 Dptr.)

Note: Do not use Dispensing guns with cracks, fractures, holes or similar.

Sterilization

Equipment: Steam sterilizer according to DIN EN 285 or DIN EN 13060 with type B process.

Pre-vacuum process, 132 °C and sterilization time minimum 3 min (longer holding times are possible)
(132 °C, 3 min were validated)

- 1) Place the Dispensing gun in the sterilizer chamber
- 2) Start the sterilization program
- 3) At the end of sterilization program remove the Dispensing gun and let it cool down

Undesired Side effects and events

According to EU Medical Device Regulation users/ patients are obliged to report occurred serious incidents with a medical device to its manufacturer and to the competent authority of the country, the incident occurred.

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