Instruction for use

Latex-free prophylaxis cups with lamellae for thorough cleaning and polishing. Available in different versions and hardness. The spray guard on the shoulder of the Cup effectively prevents paste from entering into the contra-angle handpiece.

- Mounted for contra-angle handpiece - Designed for mounting on the handpiece.
The shanks comply with the standard requirements of ISO 1797.
- Snap on – compatible with mandrels of type 5/ ISO 13295
- Screw in
- YELLOW: Hard, for removing severe plaque
- PURPLE: Medium, universal hardness for cleaning and polishing
- BLUE: Soft, for final polishing of the tooth surface, especially near the gingival sulcus to protect the soft tissue.

Latex free polishers are characterized by a particularly hardwearing material with a low coefficient of friction, which results in only minor warming of the tooth.

Application

Prophy Cups used with prophylactic paste. The Prophy Cups were mounted on a dental contra-angle handpiece. After applying the paste to the cup or directly to the tooth, the dental contra-angle handpiece is placed on the tooth and is set in rotating motion.

For attention

Maximum working speed: 2500 min rpm. Becht prophylactic polishers are class IIa medical devices and are intended for single use only. All packaging units are shipped non-sterile. Check by hand and by using the handpiece on air if the cup fits well. For professional use only.

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