SIMPLEE®

All you need. No more. But no less.

ASPIRATING SYRINGE

Ⓢ Description

Aspirating syringes are used in conjunction with a needle cartridge, Aspirating Syringes are available in Type CW and Type A, the syringes are reusable and are provided clean and supplied non-sterile.

Indication

Aspirating syringes are indicated to be used in conjunction with anesthetic needles and anesthetic cartridges for injection of anesthetic solutions in the oral cavity.

Warnings

1. To avoid needle sticks, re-cap needle after injection has been given.
2. To avoid leakage during an injection, ensure that needle is straight as it penetrates the cartridge diaphragm.

Precaution

1. Syringes must be sterilized prior to use.
2. Hubs cannot be removed, do not attempt to remove hub from syringe for sterilization or for any reason as damage to the syringe will occur.
3. Do not mix metals when sterilizing syringes, sterilizing syringes with non chrome-plated brass instruments may cause damage to syringe.
4. Do not use cold sterilants, as they will react to the chrome plating of syringe.

Loading Instructions

1. Retract piston rod fully.
2. Insert 1.8 cc cartridge, plunger end first, into syringe barrel.
3. Hold cartridge firmly in place, push piston or forward with a quick jab.
4. Affix needle to aspirating syringe.
5. Expel a few drops of solution to clear needle.
6. Aspirate before injecting solution after inserting needle into tissue.

After positioning the needle, pull back on thumb ring to aspirate.

Unloading Instructions

1. Remove needle from syringe.
2. Withdraw piston rod to clear cartridge for removal
3. Tilt syringe to expel to cartridge

Cleaning Instructions

1. Remove all traces of blood, saliva and anesthesia solution.
2. Remove paraffin that has accumulated on the harpoon.

Sterilization:

The instruments can be sterilized in designated container systems or general purpose sterilization containers individually packed in the standard polyethylene or Sterilization must be done by steam with fractionated pre-vacuum. Based on DIN EN 285 and DIN EN 554 is recommended at a temperature of 134 ° C , a contact time of at least 5 minutes.

Effective steam sterilization can be achieved with the following cycle: Temperature cycle type minimum exposure time

Pre-vacuum to 134 ° C - 137 ° C for 5 minutes

To avoid damage the instrument, the temperature of 180 ° C must not be exceeded.

Storage

Sterile, packaged instruments should be stored so that they are protected from dust, moisture, insects and vermin, and temperature extremes.

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