GB Instructions for use

Intended purpose

Simplee Bloc Out is a light-curing blocking-out material. The material is used for the modelling on different materials, e. g. plaster or acrylic. The material remains stable during deep drawing.

Application

1. Fabricate the stone model and trim. Remove plaster bubbles.
2. After the model is completely dry, apply Simplee Bloc Out. This is applied to the labial surfaces of the teeth to be whitened and is used later as a reservoir for the gel. Usually this involves teeth 15-25 and 35-45, but it is also possible to prepare a smaller or larger number of teeth or single teeth.

Important: Apply Simplee Bloc Out in layers approx. 0,5 mm thick and maintain a gap of approx. 0.5-1 mm to the gingival margin. Do not cover incisal edges and occlusal surfaces with gel!

3. Polymerise Simplee Bloc Out. A standard halogen polymerisation lamp (approx. 20 sec. per tooth), a plasma light-curing unit (approx. 3-6 sec./2-3 min., depending on the manufacturer) or a light oven (approx. 2-3 min., depending on the manufacturer) can be used for polymerising.

After curing, completely remove the Inhibition layer on the surface using a cloth moistened with alcohol. Do not use acetone or similar cleaning agents.

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