GB Type of application:

Tagatose-based prophylaxis powder (non-cariogenic) for the gentle removal of supra- and subgingival plaque and discoloration. Suitable for all commercially available air polishing systems.

Instructions for use:

Please observe the instructions for use and the maximum filling height of the device manufacturer.

Subgingival application:

- Removal of periodontal biofilm.
- Cleaning of implants
- For supportive periodontal therapy (SPT) after initial treatment

Supragingival application:

- Removal of supragingival biofilm and discolorations around the sulcus
- Removal of discolorations in sensitive areas.
- Polishing of the teeth, final polishing is not necessary
- Suitable for regular prophylaxis treatment

Never treat patients suffering from chronic bronchitis or asthma with an air polishing device. The patient and operator must wear eye protection. The operator must wear a mask and gloves. Use a high-performance suction device. Apply Vaseline to the patient's lips. Do not direct the powder stream at fillings, crowns and bridges as these dental restorations could be damaged. For dental use only. Keep the container closed. **Shake well before use!**

All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.

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