## **en** Instruction for use

### **Patient target group**

Children, adolescents, adults and older patients

### **Intended users**

Dentist

### **Composition**

#### Qualitative data:

Calcium sulphate, zinc oxide, ethyl cellulose

#### Quantitative data:

60 % calcium sulphate

15 % zinc oxide

### **Intended purpose of the product**

For temporary fillings.

### **Indications**

Temporary, short-term closure of dental cavities.

### **Performance characteristics of the product**

Sealing compound, self-curing.

### **Specific precautions**

None

### **Application**

Apply the required amount of Simplee temporary filling cement into the moist cavity with a filling instrument. Hardening begins after a few minutes. In exposed areas, a 15-minute rest is recommended. Deep cavities do not need to be filled completely, thereby facilitating later removal. When closing inserts, ensure that the application is as free of pressure as possible. The best way to remove the set Simplee temporary filling cement is to use a pointed, short and strong probe inserted between the dentin and the occlusion.

### **Warnings**

#### **Residual risks**

The processing instructions and precautionary measures described in the instructions for use accompanying the product must always be observed. If, despite adherence to these instructions, an adverse reaction occurs in the patient or other adverse events occur, we request that you inform us in any case, describing the accompanying circumstances as precisely as possible.

### **Side effects**

Adverse side effects are not known. Risks in case of overdose are not known.

### **Interactions**

Interaction of the product with other medical devices, medicinal products or other substances is not known.

### **Storage**

Close the container immediately after each material withdrawal.

Shelf life of the closed packaging: see label.

After opening for the first time, the preparation can be stored for at least 6 months.

The Product should be stored between 0° C - 22°C.

### **Minimum shelf life**

Do not use Simplee temporary filling cement after the expiry date.

### **Variants**

**REF 156 129** - 2 glasses at 30.0 g, white

### **Disposal**

Unused material can be disposed of with household waste.

**All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.**

**Attention**

**Warning**

**H315** – causes skin irritation

**H319** – causes serious eye irritation

**H411** – toxic to aquatic organism, with long-lasting effect

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