## en Instruction for use

### Patient target group

All those for whom Simplee Temporary Luting Cement is required for treatment, except people who are allergic to the ingredients, here especially rosin.

### Intended user

Dentist

### Composition

#### Essential components:

Resin acid and metal oxides

#### Qualitative data:

Paraffin, peanut oil, calcium hydroxide, magnesium oxide, hydroxyapatite, zinc oxide, ethanol (traces), fatty acid from coconut, rosin, silicon dioxide.

#### Quantitative data:

Calcium hydroxide 2%, rosin 45%, fatty acid 25%, zinc oxide 10%.

### Intended purpose of the product

Simplee Temporary Luting Cement is an easy-to-handle, eugenol-free cement that does not inhibit the curing of resin-based filling and luting materials. Simplee Temporary Luting Cement is indicated for allergies to eugenol. Simplee Temporary Luting Cement contains calcium hydroxide and hydroxyapatite to protect the teeth and preserve them.

### Indications

Simplee Temporary Luting Cement is a temporary bonding cement for crowns and bridges.

### Contraindications

The material contains rosin. Do not use in patients with an allergy to rosin, peanut oil or other substances contained in the product.

### Performance characteristics of the product

The product cures after mixing.

### Preparation

Allow the product to reach room temperature for 2 hours before use.

### Application

#### Type of application:

Simplee Temporary Luting Cement is processed at room temperature. Longer processing times result at low temperatures (cooled glass plate). The mixing ratio of the base (brown) and catalyst paste (white) is 2 to 1 in terms of volume (= 1 to 1 weight ratio). Therefore, the expressed strand of the base paste should be twice as long as the strand of the catalyst paste. Apply both pastes to a mixing block of non-absorbent material or a glass plate. Mix the paste for about 30 seconds until a homogeneous mass is obtained. Apply a thin layer to the inside of the temporary crown or bridge or wet die preparation. Never use on dry stumps, always

moisten them first. Simplee Temporary Luting Cement can be used for 1 to 2 minutes and cures in the mouth in 3 to 6 minutes. Remove excess material approx. 3 to 5 minutes after curing.

#### Additional notes:

Softer material and longer working times can be achieved by adding a little more Simplee Temporary Luting Cement base paste (brown) or a small amount of Vaseline. In addition, moisture (damp stumps) makes it easier to remove later. Harder and faster setting material can be achieved by adding a little more Simplee Temporary Luting Cement catalyst paste (white) or by mixing in zinc oxide. Such a harder material is better suited for multi- and large-surface cavities as well as temporary long-term closures (maximum 1 month). Residues of Simplee Temporary Luting Cement can easily be removed from crowns by treating them with a cold spray.

#### Processing instructions:

The processing instructions and precautionary measures described in the instructions for use belonging to the product must be observed in any case. If, despite adherence to these instructions, an adverse reaction occurs in the patient or other adverse events occur, we request that you inform us in any case, describing the accompanying circumstances as precisely as possible.

### Warnings/residual risks

The product contains rosin. Persons who have an allergy to rosin should not come into contact with the product. Avoid contact of mixed cement with skin or mucous membrane. If contact does occur, remove the material with cotton wools soaked in alcohol and rinse with plenty of water. In case of eye contact, rinse with plenty of water with the eyelids open and consult an ophthalmologist.

### Risks/side effects

Risks of overdose are not known. Adverse side effects are not known.

### Interactions

Interaction of the product with other medical devices, medicinal products or other substances is not known.

### Storage

The Product should be stored between 0° C - 22° C.

Close the tubes well after use.

Do not interchange closures.

### Minimum shelf life

Do not use Simplee Temporary Luting Cement after the expiry date.

### Variants

**REF 156 128** - 25,0 g base paste in tube,  
25,0 g catalyst paste in tube

### Disposal

Uncured material should be disposed of as hazardous waste.

**All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.**

Warning

Danger

Warnin

H315 – causes skin irritation

H318 – causes serious eye damage

H410 – very toxic to aquatic life with long lasting effects

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