

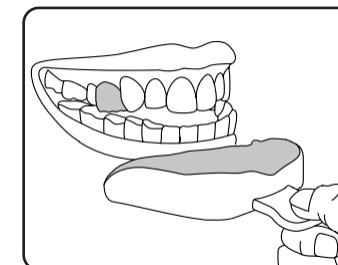
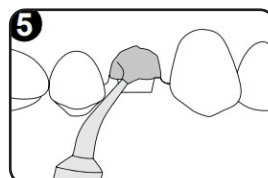
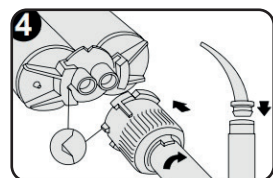
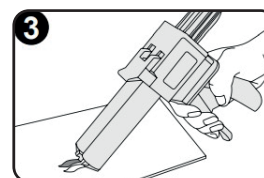
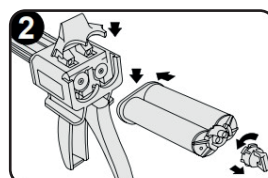
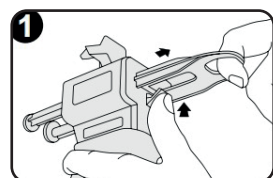
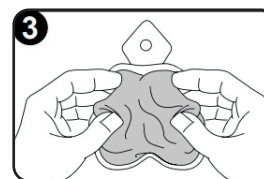
- (GB) Instruction for use**
- (DE) Gebrauchsanweisung**
- (FR) Mode d'emploi**
- (IT) Istruzioni per l'uso**
- (CZ) Návod k použití**
- (HU) Használati útmutató**
- (SE) Bruksanvisning**
- (NO) Bruksanvisning**
- (FI) Käyttöohje**
- (DA) Brugsanvisning**
- (EE) Kasutusjuhend**
- (LV) Lietošanas pamācība**
- (SI) Navodila za uporabo**
- (LIT) Naudojimo instrukcija**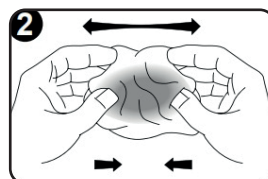

Made in Germany

Medical Device/ Medizinprodukt/ Dispositif médical/ Dispositivo medico/
Zdravotnický prostředek/ Orvosi termék/ Medicinteknisk product/
Medisinsk utstyr/ Lääkinnällinen laite/ Medicinsk udstyr/ Meditsiinitoode/
Medicīnas ierīce/ Medicinski pripomoček/ Medicinos priemonė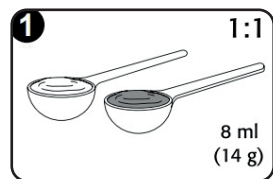

Elastomeric precision impression material on vinyl polysiloxane basis, addition-curing.

Thank you for the confidence you have shown in our company. You have made the right decision with the Simplee Silic A impression system. Please always read these instructions for use through carefully before using the product! Simplee Silic A products are only intended for use in dental applications by correspondingly qualified professionals. The medical device must be employed responsibly and with due caution.

Product description

Simplee Silic A is an impression material which satisfies the highest demands. The putty-wash materials offer very high thixotropism, outstanding resilience and hydrophilic properties. All Simplee Silic A materials have a thermo-activating formulation which offers the user a variable processing time and thus the greatest processing comfort. The time in mouth ist patient-friendly as it always equally short. The system is rounded off by the highly contrasting colour system and the pleasant mild mint flavour.

Indication

Crowns and bridges, partial crowns, inlays, onlays, implants, transfer impressions, functional impressions.

Impression technique

- **Simplee Silic A putty/ putty soft:** Preliminary impression material for the putty-wash technique (2-step), situation impression.
- **Simplee Silic A medium/ light/ light fast:** wash material for putty-wash technique and double mixing technique.

Dosage/ processing (see also illustration)

Simplee Silic A putty materials for manual mixing applications

Remove the required amount of Simplee Silic A putty with the respective measuring spoon in a mass ration of 1:1 of the catalyst an base. Combine the two components, draw the putty out to form a small strand, then fold it together again and knead it with your fingertips. Repeat this process a few times until you achieve a homogeneous colour, but always complying with the minimum mixing time of 30 seconds. Close containers tightly immediately after use and do not interchange the lids and measuring spoons.

Simplee Silic A materials in 50ml safety cartridges

Insert the cartridge in the 1:1 mixing gun, remove the end cap and dispense material until material is flowing evenly from both openings. Now place the mixing tip (single-use) over the openings (ensure that the colour code of the end cap and mixing tip are identical) and ensure that it is positioned correctly. To close it, turn the tip 90° clockwise. If necessary,

use an intraoral tip (single-use). Operate the mixing gun again and dispense material in the desired amount. The used mixing tip remains on the cartridge and serves as a cap. Prior to the next use, remove the used mixing tip, check the opening for a plug that forms in isolated cases, clean if necessary and proceed as usual.

Impression procedure

When using a suitable tray adhesive, please follow the manufacturer's instructions for use. After inserting the tray with the Simplee Silic A materials into the patients mouth, press it into place briefly and then hold in place until fully cured. Always check the material intraorally to assess the degree of curing. When using the putty-wash technique (2-step), the first impression must be cleaned and dried in order to ensure cross-linking with the wash material.

Disinfection

The impressions can be disinfected with a 2% glutaraldehyde solution after removing the impression from the mouth, rinse it under running water for 15 seconds. Then disinfect the impression by immersing it in the disinfectant thoroughly. Observing the disinfectant manufacturer's specifications is essential. We recommend (aldehyde-free) products.

*Further information ca be found at www.dgzmk.de, and in *J.Prosthet.Dent.* 1999 may, 81 (5), 621.*

Electroplating

The impressions can be electroplated with copper or silver in standard baths.

Model fabrication

The impression can be poured after just 60 mins. There are no other time restrictions. All class 3 and 4 gypsums which comply with the requirements in ISO 6873, can be used. Undamaged impressions can be poured several times.

Working and safety instructions

The products described above must only be used in accordance with their instructions for use. The user has sole liability for any other use which contravenes these instructions for use.

- Close containers tightly immediately after use and do not interchange the lids and measuring spoons.
- Products marked „single use“ are intended for a single use. To avoid cross-contamination, disinfect cartridges and mixing guns before the next use, e.g., with wipes.
- Solutions used prior to impression taking, such as mouth rinses or retraction solutions, may interfere with the setting reaction and must be removed thoroughly.

- Contact between the material an certain types of gloves, e.g. latex gloves, should be avoided as the catalyst can be damaged. We recommended nitril/vinyl or PE-based gloves.
- Do not leave any impression material residues in the oral cavity.
- Avoid contact with clothing, as cross-linked silicones are chemically resistant and leave stains which cannot be removed.
- Do not use Simplee Silic A products with polyether, polysulphides or condensation-curing silicones.
- To prevent possible incompatibilities, do not combine Simplee Silic A with A-silicones from other manufacturers.
- Storage: store in a dry place at between 5°C and 27°C (41 °F and 80,6°F).
- Shelf life: See date printed on package/ label. Do not use after expiry date.
- Processing temperature: Room temperature beetween 18°C and 25°C (64,4°F to 77°F).

Contraindication

The product must not be used in patients with known allergies to flavourings or other ingredients (see safety data sheet or ask the manufacturer). This should be clarified before use!

Technical data and product features

Technical specifications	Simplee Silic A putty/ putty soft	Simplee Silic A medium	Simplee Silic A light/ light fast
Consistency/ EN ISO 4823	Kneadable/ Type 0	Medium-bodied/ Type 2	Light-bodied/ Type 3
Colour	Light green/ Blue	Lilac	Pink
Dosage (1:1)/ mixing time	30 sec.	Automatic	Automatic
Working time incl. mixing time*	Up to 2 min.	Up to 2 min.	Up to 2 min. / up to 1 min. (fast)
Time in mouth	> 2 min.	> 2 min.	> 2 min./ > 1 min (fast)
Total setting time	4 min.	4 min.	4 min. / 2 min. (fast)
Deformation under pressure (approx.)	2,0 %	2,0 %	3,0 %
Recovery from deformation (approx.)	99,4 %	99,6 %	99,6 %
Linear dimensional change (approx.)	0,20 %	0,13 %	0,16 %
Flavour	Mint	Mint	Mint

* At 23 °C/ 73,4 °F. The times sprecified in the table above are based on a relative humidity of 50 +/- 10%. As a general rule: higher temperatures accelerate setting and lower temperatures delay it.

Warnings

- **Eye contact:** Avoid contact with eyes. In case of contact, rinse eyes immediately with plenty of clean water and consult an ophtalmologist without delay.
- **Swallowing:** If swallowed, rinse out the mouth and drink plenty of water. Always contact a doctor.
- **Allergic reactions:** It is not possible to rule out allergic reactions in hypersensitive people. If acute allergic reactions should occur during the treatment, treatment with the product should be stopped immediately and a doctor should be consulted.

Should a serious incident occur unexpectedly in connection with this medical device, it must be reported to the manufacturer and the competent authority in your Member State.

Please also observe the warnings in the safety data sheet!

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